



What is InvisibleMi+?

InvisibleMi+ is a combined test of endometrial receptivity test beREADY (beready.ee) together with endometrial and vaginal microbiome test.

Why to use InvisibleMi+?

Successful embryo implantation is influenced by endometrial receptivity and endometrial/vaginal microbiota.

InvisibleMi+ allows women to determine the best time for embryo implantation and at the same time provides knowledge if the microbial environment in the uterus is favorable.

Competence Centre on Health Technologies
Teaduspargi 13, 50411 Tartu
ESTONIA, EU

Certified medical laboratory
(Estonian Health Board permit L05235)

invisiblemi.eu

Phone: +372 733 0403
Email: info@invisiblemi.eu

INVISIBLEmi

Vaginal microbiome sample
collection **instructions**

InvisibleMI+ for fertility
clinics

INVISIBLEmi
Making your invisible **visible**

INVISIBLEmi

making your invisible visible

Vaginal microbiome sample collection instructions

InvisibleMi+ for Fertility Clinics

InvisibleMi+ for Fertility Clinics is a combined test of endometrial receptivity test beREADY together with endometrial and vaginal microbiome test.


The test kit contains a beREADY test collection tube, an InvisibleMi+ microbiome tube with vaginal sample collection swab and a BIO-specimen bag.

Vaginal microbiome sample is collected before beREADY endometrial biopsy by using a swab with a soft white tip and a handle.

Same endometrial biopsy will be used for both beREADY test and endometrial microbiome test (no separate sample is needed). For beREADY sample collection, please follow the beREADY operation manual provided in the test kit.

1.

Take the tube out from its pouch and hold the swab at the red mark molded breakpoint on the shaft.

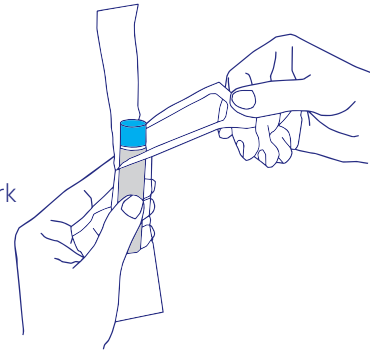


IMPORTANT:

Do NOT allow the swab tip to come into contact with body areas or surface different from the sampling site.
Do NOT touch the swab below the break point with hands.

2.

Insert the swab into the vagina. Gently rotate the swab 3 times against the sides of the vagina. Pull out the swab carefully.

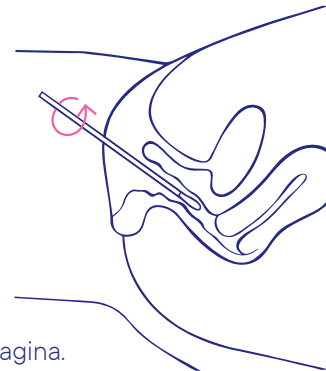


IMPORTANT:

Do NOT bend swab during sample collection as it may break.

3.

Unscrew the cap from the collection tube and insert the swab into the tube. Bend and break the swab at the breaking point. Discard the swab handle. The swab tip remains in the stabilization liquid.

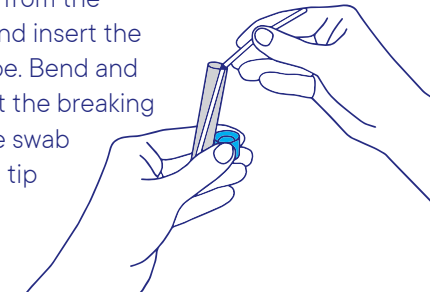


IMPORTANT:

Make sure that the tip is in the liquid. Make sure not to spill the medium.

4.

Close the cap of the tube tightly. Place the tube into BIO-specimen bag and then into the patient's beREADY test box.

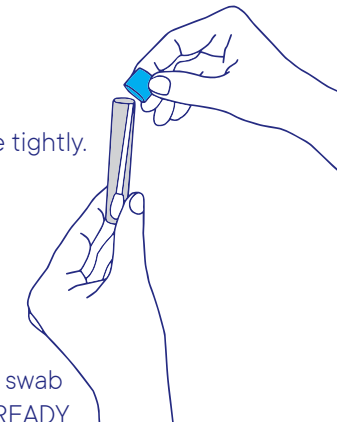

Send InvisibleMi+ vaginal swab sample together with beREADY endometrial biopsy sample.

! WARNINGS AND PRECAUTIONS !

Do NOT spill the liquid inside the collection tube. In case the liquid comes in contact with your skin wash skin with water.

Do NOT touch skin or any surfaces with the swab tip after it is removed from the package.